

SWISSEYE®

Swiss Eye International GmbH

Nord-West-Ring 14
D - 32832 Augustdorf
Phone: +49 (0) 5237/2316-0
Fax: +49 (0) 5237/2316-100

info@swisseye.com
www.swisseye.com


 FIND US ON FACEBOOK!



05.2014


5

- Sportbrille leicht biegen und Nasensteg inklusive RX Clipadapter an der Scheibe arretieren
- *slightly bend the sports glasses and attach the nose bridge including RX clip adapter to the glass*

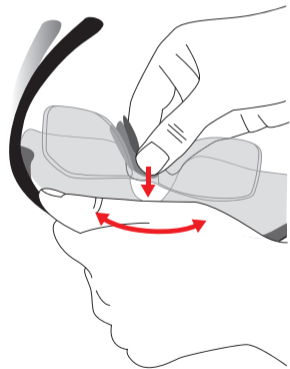

6

- Nasensteg inklusive RX Clipadapter einschieben
- *insert nose bridge including RX clip adapter*

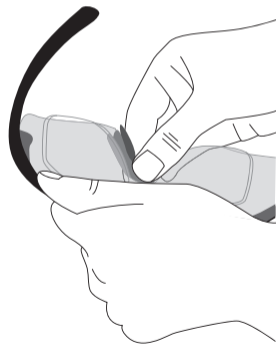


WECHSEL DES RX CLIPADAPTER
CHANGE OF THE RX CLIP ADAPTER

SWISSEYE®

1

- Leicht an einer Seite des Nasenstegs ziehen, bis dieser sich löst


- *pull softly at one side of the nose bridge until it releases*

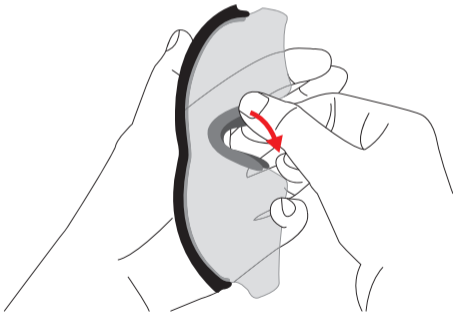

2

- Den Nasensteg abnehmen


- *now take off the nose bridge*

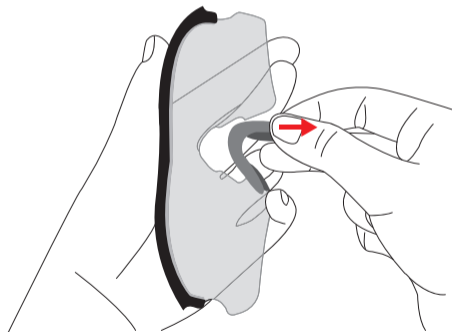

3

- Zwei Arretierungspunkte anfixieren und RX Clipadapter von oben in den Nasensteg stecken

- *aim two fixing points and put the RX clip adapter onto the nose bridge*

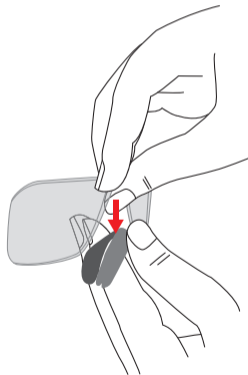

4

- Prüfen Sie den Sitz des RX Clipadapters in der Arretierung

- *control the correct placement of the RX clip adapter into the locking*

